

Dashboard | Arven - Your Medication Assitant

https://app.rx-arven.com

Status

Location: Navigating 

Connection: Strong 

Battery: 3% 

Dashboard

Alerts

Loratadine was not delivered on Jan 17th, 2023 @ 9:00am....



Battery level critically low!



View All

Medications

Loratadine

12 doses left

Ibuprofen



3 doses left

View All

Active Schedules

Loratadine

next dose @ 8:00am tomorrow

View All

Manage Medications | Arven - Your Medication Assitant

←

→

✕

🏠

https://app.rx-arven.com/medications

🔍

Rx

Medications

➕ Add Medication

Medication	Remaining	Schedules	Actions
Loratadine	12 pills	Daily: 8:00am, 8:00pm	
Ibuprofen	6 pills	M: 8:00am, W: 2:00pm, F: 8:00pm	
Tylenol	30 pills	+ Add Schedule	

←

→

✕

🏠

https://app.rx-arven.com/medications

🔍

Rx

Medications

+

Add Medication

Med

Lora

Ibup

Tyler

Schedules for Loratadine

☐

- Perform Action -

▼

Frequency

☐ Daily

▼

Time

08:00

🕒

Frequency

☐ Daily

▼

Time

20:00

🕒

📅

Add Schedule

Cancel

→

Save

→

Actions

👤

🗑

🔊

👤

🗑

🔊

👤


🗑

🔊





https://app.rx-arven.com/medications/edit/1

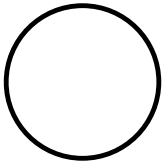

← Back to Medications →



Label

Loratadine

Dose Quantity

1

Dose Units

pill



Amount on Hand

12



Low Threshold

8



Medications

Schedules



Add Schedule



Frequency

Date

Time

Actions

Daily

-

8:00am



Daily

-

8:00pm




Last Delivered: Jan 26th, 2023 @ 8:00am





https://app.rx-arven.com/history

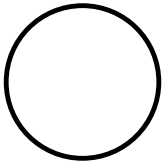

Enter a value to search by...

Filter + 01/15/2023 - 01/17/2023 X



Date ↓	Time	Source	Details
Jan 17th, 2023	11:23am	Arven	Battery level critically low!
Jan 17th, 2023	10:20am	System	Notification sent indicating medication low.
Jan 17th, 2023	9:07am	Arven	Loratadine was not delivered on Jan 17th, 2023 @ 9:00am. No signal detected for user.
Jan 16th, 2023	8:00pm	User	User added new medication: Loratadine.

Items per page: 5 1-5 of 20 << < > >>

Configuration | Arven - Your Medication Assitant

←

→

✕

🏠

https://app.rx-arven.com/configuration

🔍

⬤

Device Serial

RX-AR-20230001

📶

Locate Stairs

🔌

Reboot Device

🩺

Run Diagnostics

🔄

Factory Reset

🕒

Your Name

Johnny

Your Email

john.smith@example.com

✉

📖

Timezone

America/Edmonton (MST) -7:00

🌐

🕒

🔄

⚙

Configuration

〰

Setup | Arven - Your Medication Assitant

https://app.rx-arven.com/setup

Say Hello to *Arven*:
Your Personal Medication Assitant

Device Serial

e.g. RX-AR-2023XXXX

Timezone

America/Edmonton (MST) -7:00

Your Name

e.g. John


Your Email

e.g. john.smith@example.com

Login Password



https://app.rx-arven.com/login

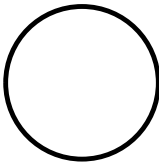

Arven missed you...
Welcome back!

Email

e.g. john.smith@example.com

Password

Sign In



Haven't met Arven yet? [Connect with Arven](#) now.

